Male External Catheter

Designed to effectively manage urinary incontinence.

Indications for Use

The Surity™ Male External Catheter is intended for non-invasive management of urinary incontinence in adults with male anatomy.

Contraindications: Users with urinary retention.

SYMBOL	SYMBOL TITLE	EXPLANATORY TEXT
	Manufacturer	Indicates the medical device manufacturer
	Lot number	Indicates the manufacturer's batch code so that the batch or lot can be identified
	Reference number	Indicates the manufacturer's reference number so that the medical device can be identified
	Medical device	Indicates that this product is a medical device
	Single use	Indicates a medical device that is intended for one single use only
	Do not use if damaged or opened	Indicates that the product should not be used if the package has been damaged or opened
	Use by	Indicates the date after which the medical device is not to be used
	Consult Instructions for Use	Indicates the need for the user to consult the instructions for use


 Manufacturer: Wellspect HealthCare
Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden

Doc. No. PD-00109993-A, 2025-12-04


Warnings and Precautions

- The Surity™ Male External Catheter is designed for single-use only. Do not wash or reuse the external catheter. Reuse may result in problems such as skin irritation or infection, malfunction, or product damage.
- Do not use this product if the user has recently undergone surgery involving the external male genitalia. Do not apply the external catheter over fresh surgical wounds.
- Do not use on irritated skin (e.g., rashes, skin tears, blisters, skin maceration, or folliculitis).
- Always inspect the application site for signs of irritation or skin breakdown before use. Clean and thoroughly dry the skin prior to applying a new external catheter.
- Do not use barrier creams, lotions, or ointments on the penis or surrounding pubic skin when using this product, as these may interfere with adhesion or reduce suction.
- To avoid potential skin injury, never pull the external catheter directly away from the user. Always remove the external catheter by gently peeling the adhesive downward, in the direction from head to foot.
- Not recommended for users who may unintentionally remove the external catheter, as this could result in skin injury or loss of suction.
- After use, dispose of the external catheter in accordance with applicable local, state, and federal laws and regulations.
- If an allergic reaction occurs, stop use immediately and contact your doctor.
- If any serious incident arises from the use of this product, contact the manufacturer and national competent authority.

Male Anatomy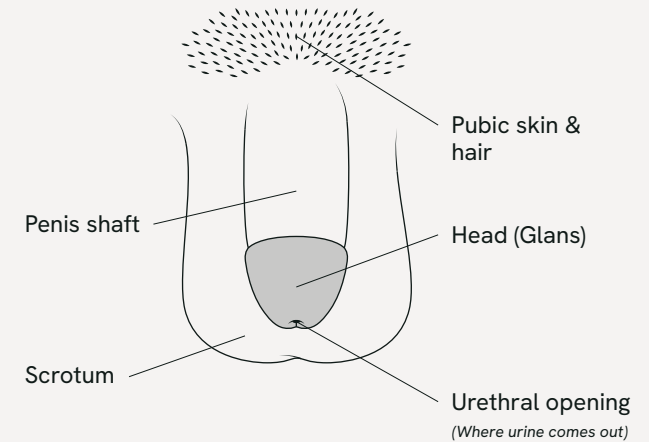

Male External Catheter

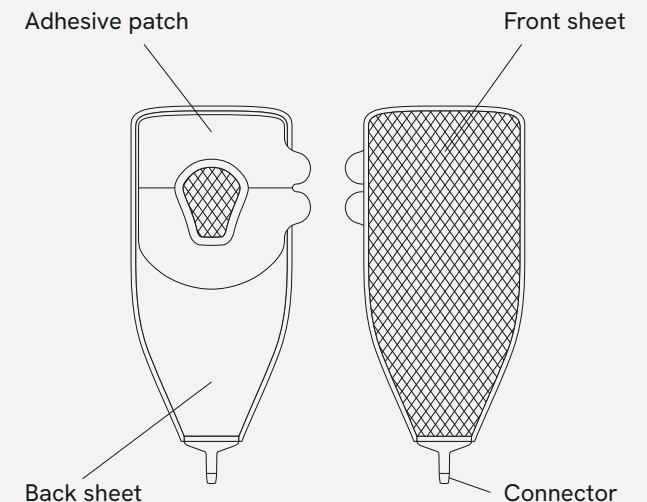Instructions for Use

Usage

- For home use only.

How to Prepare

- Wash your hands before and after placing or removing the Surity™ Male External Catheter.
- Wear gloves if you're placing or removing this product for someone else.
- Trim or clip the pubic hair to ensure the external catheter securely adheres to the skin. (Fig. A)
- Clean the penis and surrounding pubic skin with mild soap and water or wet wipes, then dry thoroughly before placement. (Fig. A)
- Do not use if the user has visible signs of skin irritation or breakdown in the genital area or at the application site.
- If the user has frequent episodes of bowel incontinence, make sure that a fecal management system is in place.

Urine Collection System Setup

- Assemble your at-home urine collection system according to its manufacturer instructions.

External Catheter Placement







1. Make sure the user is lying on their back or in a reclined position with legs separated.
2. Position the external catheter over the penis until it reaches the base of the penis shaft. Center the penis inside the opening. NOTE: Do not place the scrotum inside the external catheter. (Fig. B)
3. Gently remove the adhesive lining from the back of the external catheter by grabbing the tabs and carefully peeling each section off to the side. Press the adhesive against the skin at the application site. Ensure the adhesive fully adheres to the base of the penis shaft. (Fig. C)
4. Securely connect the Surity™ Male External Catheter tubing to the collection system tubing, ensuring a snug and leak-free connection. (Fig. D)
5. Turn on the urine collection system by pressing the power switch.
6. Allow the legs to return to a natural, closed position while ensuring the external catheter stays in place. (Fig. E)

External Catheter Removal

7. Make sure the user is lying on their back or in a reclined position with legs separated.
8. With the suction still on, gently lift the top edge of the adhesive off the skin. In a slow, downward peeling motion, remove the external catheter. (Fig. F)
9. Disconnect the external catheter from the urine collection system tubing.
10. Dispose of the used external catheter in accordance with local, state, and federal waste disposal guidelines. (Fig. F)
11. Turn off the urine collection system by pressing the power switch.

Maintenance and Replacement

- Replace the external catheter at least every 24 hours, or immediately if contaminated with feces, blood, or semen.
- Always check the skin condition and clean the genital area before placing a new external catheter.
- Store the products in room temperature, 60-77°F.

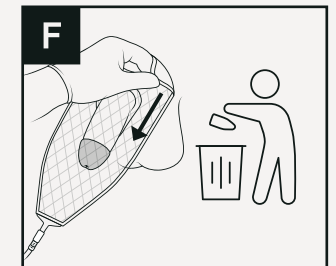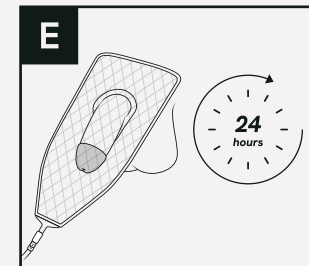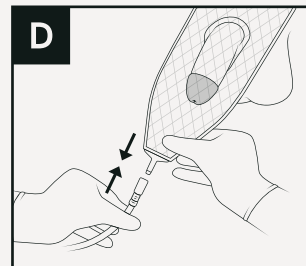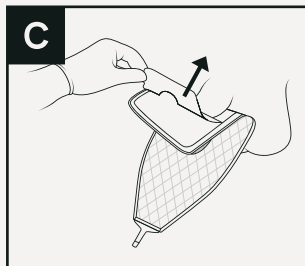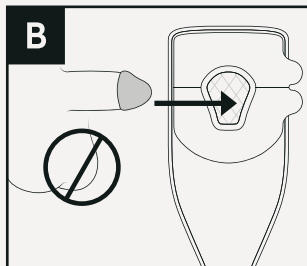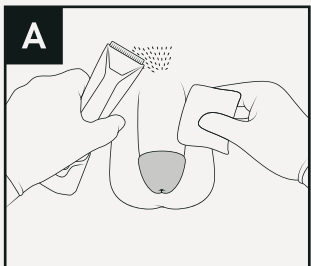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