

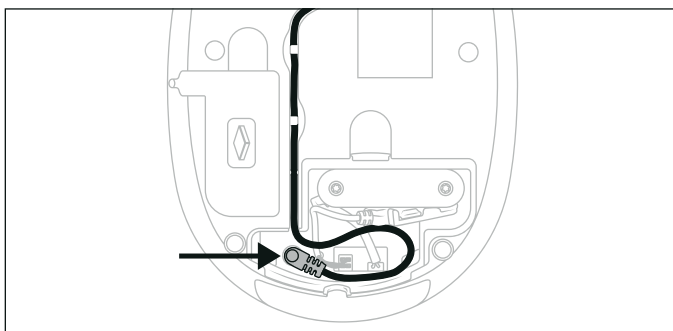
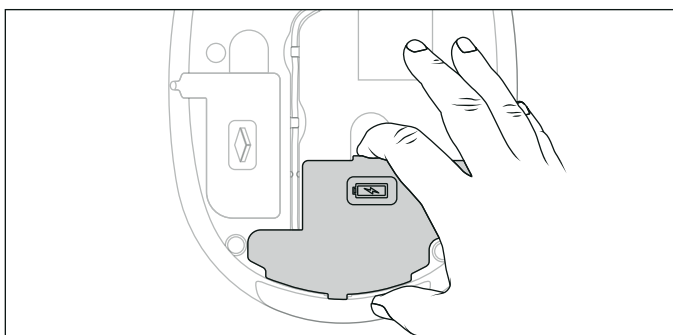


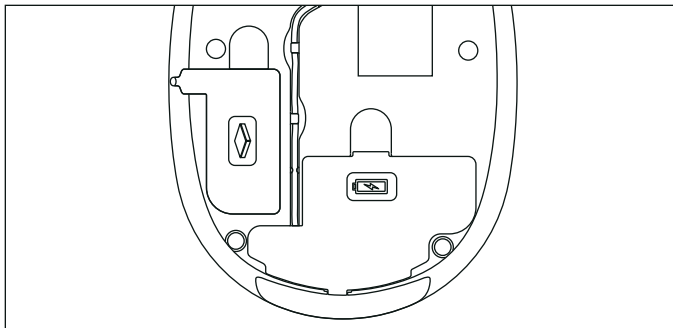





Power Adapter Installation Guide

 **Read the Surity[™] Urinary Management
System Instructions For Use before use.**

1. Disconnect the power adapter from the electrical outlet.
2. Turn the suction base upside down to reveal the compartments.
3. Open the power compartment lid by gently pressing a fingertip into the notch and lifting.
4. Unplug the power adapter's device connector.

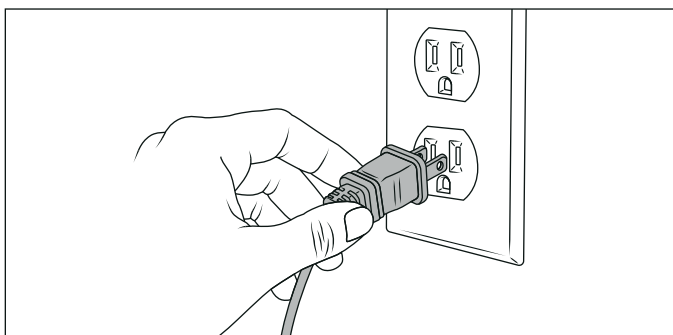

5. Connect the new power adapter and arrange the cable in the same way as before.
6. Close the power compartment lid by fitting the tabs into the openings on the suction base. Gently press down until it clicks into place.
7. Gently press the power cable into the cable holder at the bottom of the suction base. This will help keep the power cable secure and prevent it from making the base unstable.
8. Connect the power adapter to an electrical outlet.

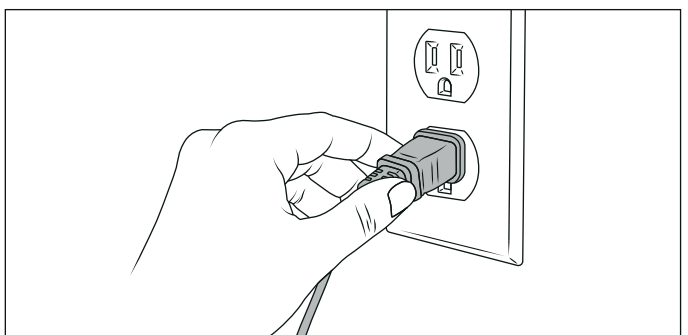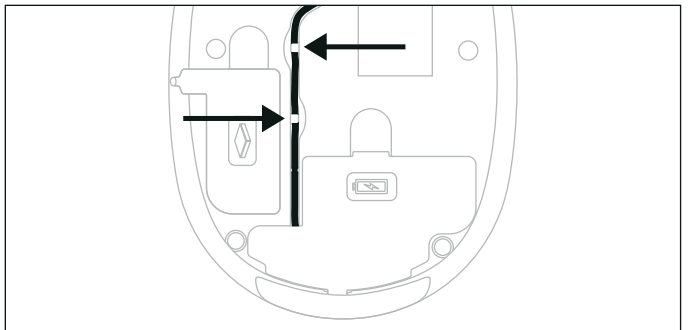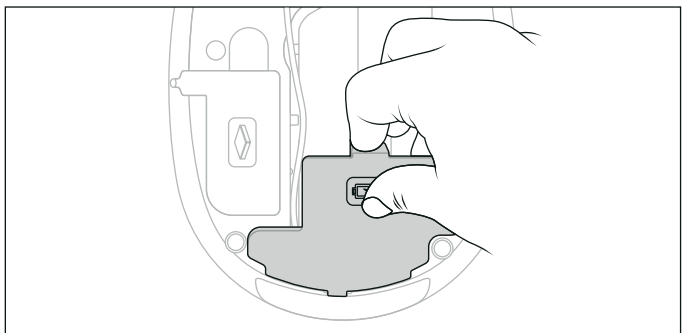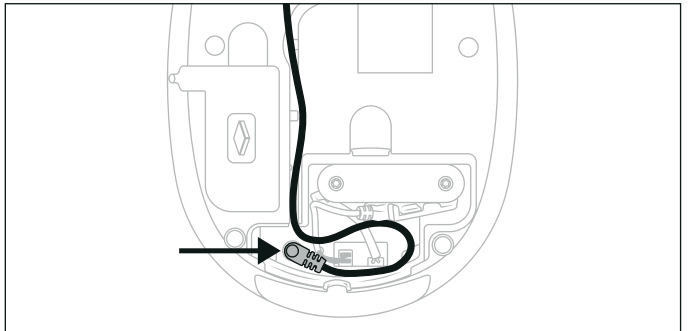

If you need any help, don't hesitate to contact us using the information below.



www.surity.care



866-324-2072



support@surity.care



Wellspect HealthCare
Aminogatan 1, P.O. Box 14,
SE-431 21 Mölndal, Sweden